USER'S GUIDE

Prenatal Heartbeat Monitor

Model: Roo

The Features described in this user's guide are subject to modifications without prior notice.

Welcome...

to your new Roo Prenatal Heartbeat Monitor!

Thank you for purchasing your new Roo. The Roo portable prenatal listening system is easy to use. It is a completely safe and non-invasive way to listen to your unborn baby. Unlike ultrasound, it emits no energy into you or your baby.

Using the Hubble's app's innovative sound tracking software, Roo lets you listen to your unborn baby's heartbeat and share it with friends and family. As your pregnancy progresses, the Hubble app will offer regular tips and information to help you prepare for your baby's birth.

Please keep your original dated sales receipt for your records. For warranty service of your Roo product, you will need to provide a copy of your dated sales receipt to confirm warranty status. Registration is not required for warranty coverage.

For product-related questions, please contact:

E-mail: support@Rooprenatal.com.


On the Web: www.Rooprenatal.com.

This User Guide has all the information you need to make the most out of your product. Please read the Safety Instructions on page 6 before you install the unit.

Inside the Box

- Prenatal heartbeat monitor
- Positioning belt
- Micro USB / USB cable
- Pouch
- Quick Start Guide

Overview of the Roo

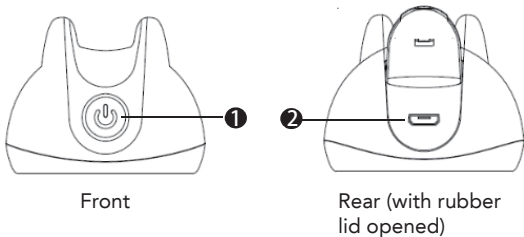1. POWER
2. USB power socket

WARNING

The Roo Prenatal Heartbeat Monitor is NOT a medical device. It should NOT be used or relied upon for diagnostic or other medical purposes. The product should not be used in place of regular prenatal care by your physician.

Table of contents

1. Important Safety Information.....	5
2. Setting up your Roo	7
2.1 Power On your Roo	7
3. Minimum System Requirement	8
4. Connect Roo to your Android™ / iPhone® / iPad® Devices ...	9
5. Operating your Roo	13
6. Disposal of the Device (Environment).....	16
7. General Information	17
8. Care and Cleaning	19
9. Troubleshooting	20
10. FCC and IC Regulations	22
11. Technical Specifications.....	23

1. Important Safety Information

To obtain optimal user experience, please read this manual carefully prior to use.

- The product is intended for your general knowledge only and is not a substitute for professional medical advice or treatment for specific medical conditions.
- You should not use this product to diagnose or treat a health problem or disease without consulting with a qualified healthcare provider.
- Please consult your physician with any questions or concerns you may have regarding your condition.
- Do not be alarmed if you are not able to hear your baby's heartbeat and movements right away.
- The optimal time to listen to your baby's heartbeat is a couple of hours after eating.
- The system will usually begin to detect baby sounds at the beginning of your third trimester – typically around 24 weeks.
- Do not put the product into water, it will damage the Roo sensor.
- The device must not be used close to any source of vibration (e.g. fans, pets etc.) as the vibrations could affect the judgment of the device.
- Do not dispose of batteries by fire. When placed in a fire, the battery is likely to explode.
- Do not store the device under extreme temperature (below -20°C / -4°F or over 55°C / 131°F) or extreme humidity conditions (below 15 % RH or over 90 % RH). Failure to store device under proper conditions may cause inaccurate readings.
- Do not use if the device is damaged. Continued use of a damaged unit may cause injury, inaccurate results or serious hazard.
- Do not disassemble, repair or change any parts of the unit.


- All materials intended to have contact with users have passed the ISO 10993-5 and ISO 10993-10 standards test, whereby they are non-toxic and non-allergenic, and do not produce irritation reactions. However, the state of current scientific knowledge and technology being incomplete, other potential allergic reactions may not yet be known. Should you have an allergic reaction to the materials, please stop treatment immediately and consult your physician.
- Store the product in a dry, clean place. Do not expose the product to chemical solvents, lint, dust, direct sunshine or high temperature.
- Do not maintain or service the device while it is in use.
- Should you experience any problems with this device (e.g. set-up, maintenance or use), please contact Roo customer service or your local retailer.
- Should any unexpected operations or events occur, please report them to the manufacturer.

2. Setting up your Roo

- Charge the Roo acoustic fetal heart rate monitor with the provided USB power cable in the USB socket under the rubber lid.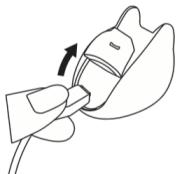


- It is recommended to charge at least 16 hours before initial use.
- When the battery power is too low, the LED indicator will flash white for few seconds and turn off. The App will show not paired.

Note:

Only use the power adapter provided.

2.1 Power On your Roo

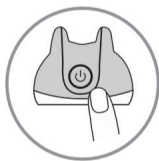- Power On - Short press the power button at front of the ROO, the LED indicator will flash white until paired with mobile.
- Power Off - Press and Hold the power button over 10 seconds, the LED indicator will be go off.

3. Minimum System Requirement

Smartphones / Tablets Minimum System Requirements:

- iOS® 7.1 or above
- Android™ 4.4 or above
- Bluetooth 4.0 or above

For more information, please visit:

<https://hubbleconnected.com/requirements>

4. Connect Roo to your Android™ / iPhone® / iPad® Devices

Download the “Hubble Connected” app

- Go to the Google Play™ Store to search for “Hubble Connected” app.
- Download the “Hubble Connected” app (or scan the QR code below) and install it on your Android™ device.

- Go to the App Store to search for “Hubble Connected”.
- Download the “Hubble Connected” app and install it on your iPhone® / iPad®.

- Run the “Hubble Connected” app on your Android™ device / iPhone® / iPad®.
- Tap SIGNUP and follow the in-app instructions to create your account. By signing up, you agree to our Terms & Conditions.

Note:


If you already have a Hubble account, tap LOGIN, enter your account information and proceed to the next steps.

- Launch the set-up wizard and follow the steps to create a user profile and add the Roo.
- Make sure that the Hubble app is active and that Bluetooth on your mobile device is activated when pairing is in progress.
- Run the “Hubble Connected” app, tap SIGNUP and follow the in-app instructions to create your Hubble Account. By signing up, you agree to our Terms & Conditions. (Pictures A1 & A2)

A1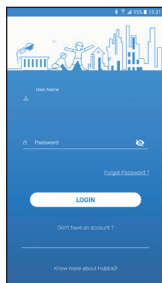


A2

Note:


If you already have a Hubble account, please tap LOGIN, enter your account information and proceed to the next steps.

- After login, tap the “+” on the screen to add new device (picture A3), select Prenatal and then select ROO (picture A4) then tap on the RoO to enter set-up mode.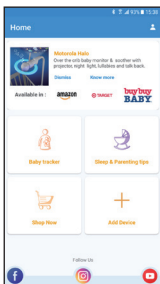


A3

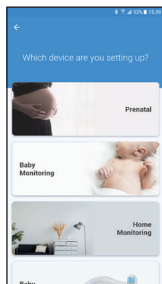
A4

- Make sure your RoO is switched on and then press “Continue” on screen. (Picture A5)
- Switch on the Bluetooth (through the APP only) connectivity on your smart device.
- Pair the ROO through the APP with mobile device. (Picture A6)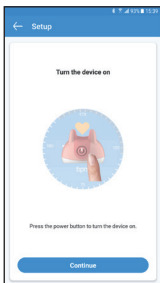


A5

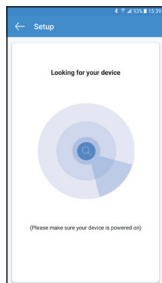A6

- You have now successfully paired the Roo with your smart device. Tap "Continue".
- Follow the steps to create the Baby Profile.
- You can now log in to your account to track your baby's heart rate and other Hubble add-on features.

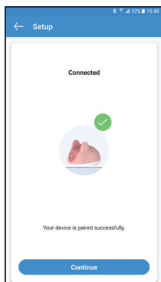
A7

5. Operating your Roo

- Follow the illustration above / below? to wrap the Positioning Belt around the abdomen reasonably tight. The top of the belt should be about 1-2 inches below the naval.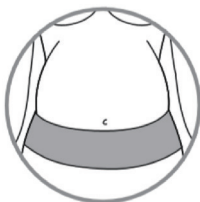


Note:

The positioning belt must be worn when using the device. Proper use of the positioning belt can increase the probability of hearing the baby's heartbeat.

- Get into a reclining position in bed, on a sofa, or in a recliner with your upper body elevated by pillows. Put the Roo device in the middle of your stomach and the positioning belt. The rubber side of the device needs to be positioned completely on the abdomen and placed inside the positioning belt.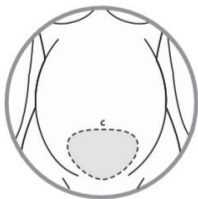

Note:

The best time to listen is 3-4 hours after eating and when you are relaxed.

- Open the Hubble app and follow the in app instructions to read the measurement, then listen to your baby's heartbeat sound. The normal heart rate of the fetus is between 110 and 180 beats per minute.
- If there is no signal obtained over 50 seconds, the app will pop up retry request.

The baby's dynamic level will affect the heart rate. In order to reduce interference, please try not to touch the device when using it. If you can't find the heartbeat, don't worry, sometimes it's normal to be a bit difficult. Familiarity with the process will gradually improve. The position of the baby, the dynamics and the distance of the fetus from the placenta can affect the effect.

6. Disposal of the Device (Environment)

At the end of the product life cycle, you should not dispose of this product with normal household waste. Take this product to a collection point for the recycling of electrical and electronic equipment. The symbol on the product, User Guide and / or box indicates this obligation.

By taking end-of-life products to a recycling point, you are ensuring that some of the product materials can be reused. By reusing some parts or raw materials from used products, you make an important contribution to the protection of the environment.

Exhausted batteries do not belong with household waste. Dispose of batteries in accordance with the current federal, state and local regulations. As a consumer, you are legally obligated to return used batteries.

Notes:

1. In environments where there is a high frequency of electrostatic discharge, the unit may malfunction and have to be reset by the user.
2. Dispose of used batteries with care; please consult retailer for details.

Please contact your local authorities should you require additional information on the collection points in your area.

7. General Information

IMPORTANT:

Satisfactory evidence of the original purchase is required for warranty service.

Please refer to www.Rooprenatal.com for any details or warranty durations offered in addition to those contained below.

Warrantor: The warrantor is Binatone North America Inc.

Terms of Warranty: Binatone North America Inc. warrants to the original retail purchaser only that the Roo (“the Product”), will be free from defects in materials and craftsmanship for the duration of the warranty period, subject to the limitations and exclusions set out below.

Warranty period: This warranty to the original retail purchaser is only valid in the original country of purchase for a Product first purchased either in the United States and will expire one (1) year from the date of the original retail sale.

If a warranty claim is made, this warranty will not apply if the Product is found by Binatone to be:

- (A) Damaged or not maintained in a reasonable manner or as recommended in the relevant Roo User Manual;
- (B) Modified, altered or used as part of any conversion kits, subassemblies or any configurations not sold by Binatone North America Inc.;
- (C) Improperly installed contrary to instructions contained in the relevant User Guide.
- (D) Repaired by someone other than an authorized Binatone Repair Agent in relation to a defect or malfunction covered by this warranty;
- or (E) Used in conjunction with any equipment, parts or a system not manufactured by Binatone.

Statement of Remedy: If the Product is found not to conform to this warranty as stated above, the Warrantor, at its discretion, will either repair the defect or replace the Product without any charge for parts or service. This warranty does not include any reimbursement or payment of any consequential damages claimed to arise from a Product's failure to comply with the warranty. Our goods come with guarantees that cannot be excluded under the US Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Procedure for obtaining warranty service: Depending on the country in which the Product was first purchased, if you believe that your Product does not conform with this warranty, you should deliver the Product, together with satisfactory evidence of your original purchase (such as a legible copy of the sales docket) to Binatone. Please refer to the Hubble website for address details. You should contact Hubble regarding any compensation that may be payable for your expenses incurred in making a warranty claim.

8. Care and Cleaning

- The appliance is only intended for household use as described in this manual. Any other use not recommended by the manufacturer may cause fire, electric shock or injury to persons.
- Use a clean, dry, soft cloth to wipe the external surface of the unit.
- Please do NOT immerse the device into water or other liquids in order to prevent electrical malfunction.
- Clean the device according to instructions when the product will not be in frequent use. Before storage, make sure that all the parts are clean and dry. Store the air purifier in a dry place.
- Please do NOT operate the device with a damaged cord or plug.
- Every effort has been made to ensure high standards of reliability of your device. However, if something does go wrong, please do not try to repair it yourself - contact Customer Service for assistance

9. Troubleshooting

What if the Roo has lost its Bluetooth® link with your Smart Device?

- Make sure the battery built into the Roo is fully charged.
- If your Smart Device is too far away it may be going out of range, so move the Smart Device closer to the Roo (within 10 meters).
- Make sure the device is added in the Hubble app, see if there is a need to pair the Roo and your Smart Device again.
- Use of other 2.4 GHz products, such as wireless networks (Wi-Fi® routers), Bluetooth™ systems, or microwave ovens, may cause interference with this product. Keep the Roo at least 5ft (1.5m) away from these types of products, or turn them off if they appear to be causing interference.

Why did my Smart Device sync during my first use, but now it won't sync or give any activity?

- Make sure Hubble app is opened when you want to read the activity.
- Remove your Roo from the app. Re-pair the device with your Smart device by adding the Roo back to the app.

What is the best period for optimal detection?

The system will usually begin to detect baby sounds at the beginning of your third trimester – typically around 24 weeks. At this stage of your pregnancy, adjust the position of the Roo sensors between your navel and pubic bone.

I can feel my baby is moving however the device is not able to read the heartbeat?

- Your baby is quite active and moves often. It is not unusual for your baby to move into a position where the heartbeat cannot be heard, especially during the earlier months of your pregnancy. If your baby moves while you are listening, simply change the position of the Roo or try again later. If your baby is particularly active, the kicking sounds may make it difficult to hear the drumming of the tiny heartbeat.
- Your weight before and during pregnancy may affect how early the system can detect your baby's heartbeat. A slender mother will generally hear the baby's heartbeat earlier in her pregnancy. This is a passive listening device and, therefore, it has no means of penetrating your body's insulating layer of fat. Mothers with a larger percentage of fat may have to wait a few more weeks until the baby's heartbeat becomes strong enough to be heard.

How long does the Roo battery last and how can I charge it?

The battery usually lasts for 5 days and charging is easy. Simply plug the USB charging cord into the Roo; a full charge usually takes 10 hours.

What else can I use the app for?

With the "Hubble Connected" app, you can access a free Prenatal Tracker to conveniently log and keep track of different activities such as water intake, kicks, a bump tracker and weight tracker. There's also a section in the app with pregnancy tips from leading organizations and answers to frequently asked questions.

I still have a question about my Roo, where can I get help?

Please contact our customer support team at:

E-mail: support@Rooprenatal.com.

On the Web: www.Rooprenatal.com.

10. FCC and IC Regulations

Warning: Changes or modifications to this equipment not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

Increase the separation between the equipment and receiver.
Consult the dealer or an experienced radio / TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Industry Canada. This device complies with the Industry Canada license-exempt RSS standard(s): Operation is subject to the following Two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

11. Technical Specifications

Technology	Bluetooth Low Energy (4.0)
Power Rating	3.7V 250mA Li-battery powered
Weight	About 170g
Storage and transportation conditions	-20° C to 55° C (-4° F to 131° F) with a relative humidity of 15 % to 90 %
Maximum separation distance	10 m (in the open environment)
Battery service life	5 days based on a usage frequency of 2 times / day

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All rights reserved.
International Limited